

# Using your stairlift

Model: **600 Siena**



**Stannah**

## Service Log

**It is important to us that our stairlifts always help you travel up and down your stairs in the safest possible way. With this in mind, we recommend our stairlifts are serviced regularly by a Stannah trained service technician. 12 Month intervals for private residence / 6 month intervals in locations other than private residence.**

[illegible]

A stairlift is a device to transport a seated person up and down the stairs safely.

North America – Standard A18.1-10 must be used for inspection and test.

# Welcome to your Siena User Guide.

To help you get the most from your new stairlift, please take a few minutes to read this guide. Then, please keep this guide in a safe place in case you need to check any details in the future. If you feel unsure about any aspect of your stairlift, please do not hesitate to get in touch with your stairlift supplier, whose details are on the back page.



Siena

**Thank you for choosing Stannah**

## Contents

<b>Getting to know your stairlift</b>	<b>4-6</b>
<b>Using your stairlift</b>	<b>8</b>
Power key	8
Unfolding the chair	9
Retractable rail	10
Seat belt	11
Arm control	12
Swiveling the seat	13
Getting off the stairlift	14
<b>Changing the batteries</b>	<b>15</b>
<b>Cleaning your stairlift</b>	<b>16</b>
<b>Safety recommendations</b>	<b>16</b>
<b>HELP</b>	<b>17</b>

Carriage Serial No: \_\_\_\_\_




You will find important safety information wherever you see the **SAFETY NOTICE** symbol shown above.

**Important:**  
**Your stairlift is not designed to operate in the event of a fire and should not be depended on as a means of escape.**

# Getting to know your stairlift

## Siena stairlift

shown in traveling position

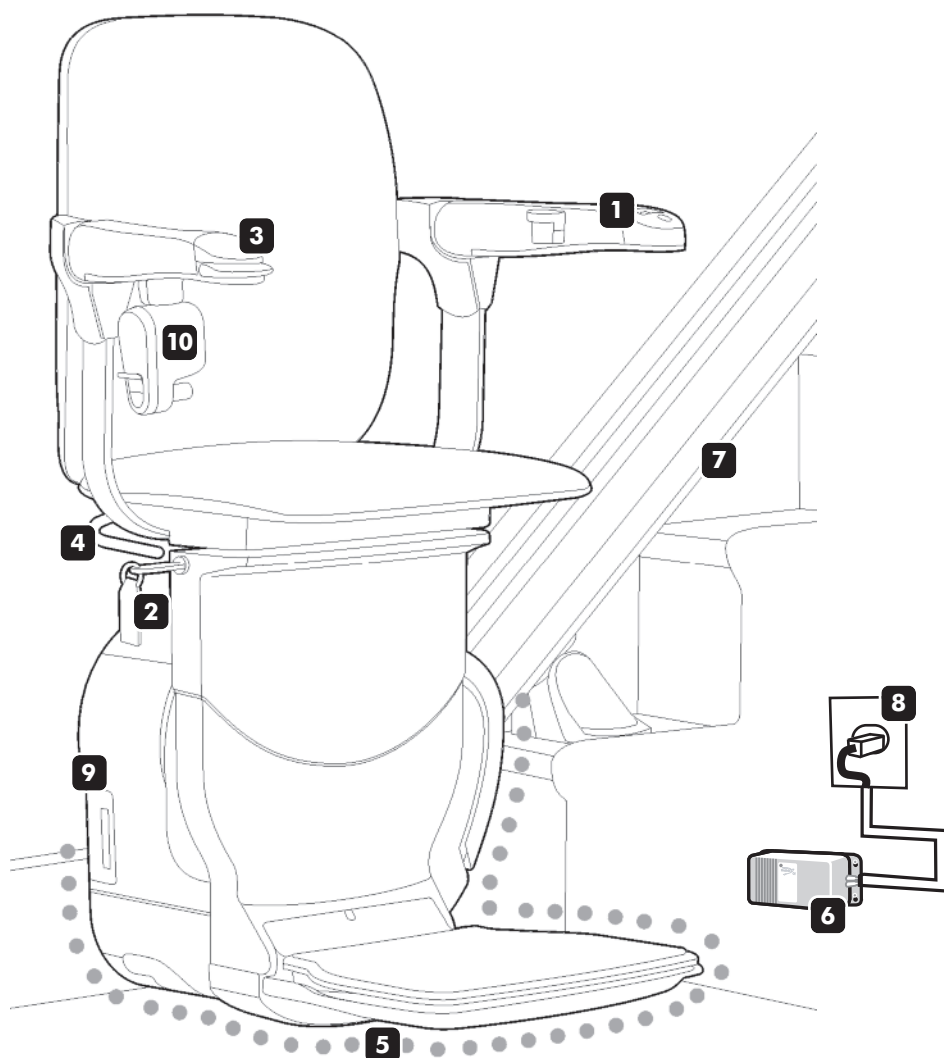

# Getting to know your stairlift

## 1 On/Off switch & power light

The power light comes on when the switch is in the on position.

## 2 Power key

The keyhole is on the side of the chair. The key must be fully inserted before the stairlift will work – see page 8.

*Please note that you can remove the power key to stop others from using your stairlift.*

## 3 Arm Control

The arm control moves the stairlift in the direction you want to travel. If your chair has a powered swivel seat, you can also use the control to swivel the seat.

## 4 Swivel levers

These are on both sides of the seat. The seat will swivel towards the landing at the top of the stairs to help you get on or off the stairlift.

## 5 Safety edges

The safety edges are there to stop the stairlift if they come into contact with any obstacles. This stops anything from becoming trapped while the stairlift is being used. See page 19 for more information.

## 6 Battery charger

Charges the batteries so the stairlift is always ready to travel. This may be positioned at either the top or bottom of your staircase.

## 7 Rail

## 8 Plug

This should normally be left plugged in. See page 6 for more information.

## 9 User display panel

Located on the side of the stairlift, it allows you to see the status of your stairlift - see page 18.

## 10 Seat belt

There are various types of seat belt. The one shown is a retractable lap belt.

**For the 'HELP' section see page 17**


## Master power switch and wall switch

The master power switch is on the back of the base of the chair and should normally be left on.

### **The stairlift will not work if this switch is off.**

If you are going away for a longer time – perhaps if you go away on vacation – you can switch off the master power switch and unplug the charger.

*Make sure that you switch off the master power switch and unplug the charger – this will ensure that the batteries will still be charged when you return.*

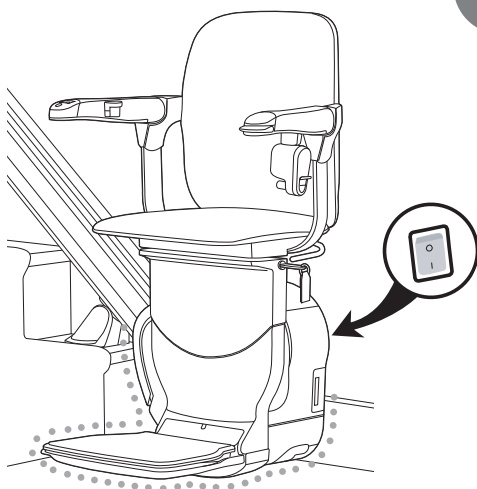

## Options

You may have chosen from the range of options we provide for the Siena stairlift.

The options we've listed below are all covered in this guide, but if you need more information about any options that are not covered, please contact your local Stannah supplier.

### **Swivel seats**

- Manual swivel seat.
- Powered swivel seat.
- 2-way powered swivel seat.


### **Seat belt**

- Immobilizer seat belt.

### **Retractable rail**

- Retractable rail.

Master power switch

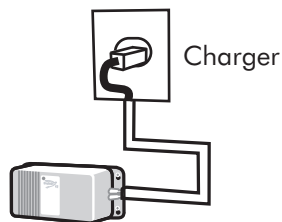

**For the 'HELP' section see page 17**

# Using your stairlift

## Remote controls

Positioned at the top and bottom of the staircase, the remote controls let you call the stairlift if it is at the opposite end of the staircase to you.


*The remotes will not work unless the stairlift is turned on.*

The remotes are battery powered.

## To move the stairlift along the rail

Press and hold the arrow button on the remote control to move the stairlift in the direction you want. You can move the stairlift whether the chair is folded or unfolded.

If you release the arrow button, the stairlift will stop and will not start again for a few seconds. Wait for a few seconds before starting the stairlift again.

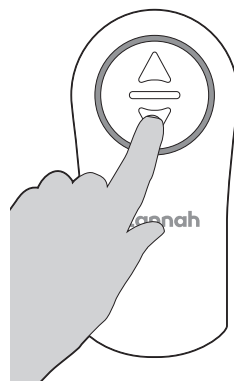

## Battery status

When you press one of the arrow buttons, the light on the control shows how much power is in the battery.

The batteries will last about one year with average usage.

*You will find information on how to check and change the batteries on page 15.*



## SAFETY NOTICE

Make sure no one else is using the stairlift before you call it using the remotes.

# Using your stairlift

## 1. Turn on the power



### To turn on the power

Insert the key into the keyhole on the side of the chair.

Make sure the key is fully pushed in.

*The stairlift and the remote controls will not work unless you have inserted the power key.*

*When the key is inserted and the on/off switch is in the on position, the light on the on/off switch will come on.*

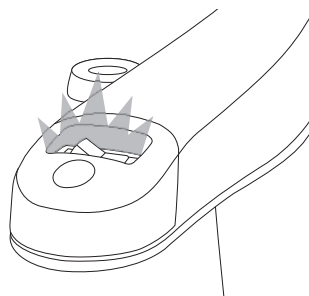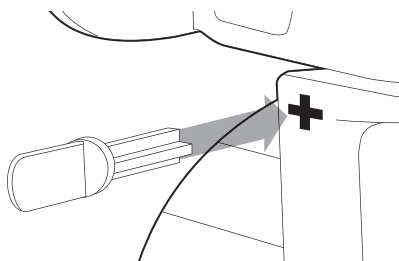


### To switch off the power

Turn the on/off switch to the off position.

Remove the key.

*If you do not want to remove the key, it can be left inserted, as shown.*

*The power light will pulse on and off.*

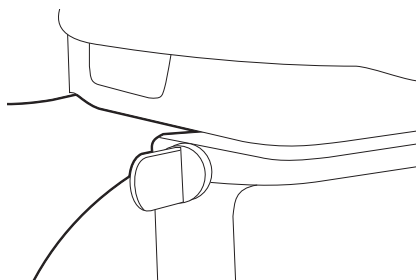

## SAFETY NOTICE

You can remove the power key to stop others from using your stairlift



# Using your stairlift

## 2. Unfolding or folding the chair

Your chair can be folded away when it is not in use to give you more space around the top and bottom of your stairs.

### To unfold the chair


Lower the seat.

*The footrest will unfold with the seat. Lower both armrests.*

### To fold the chair

Raise the seat.

*The footrest will fold with the seat. Raise both armrests.*

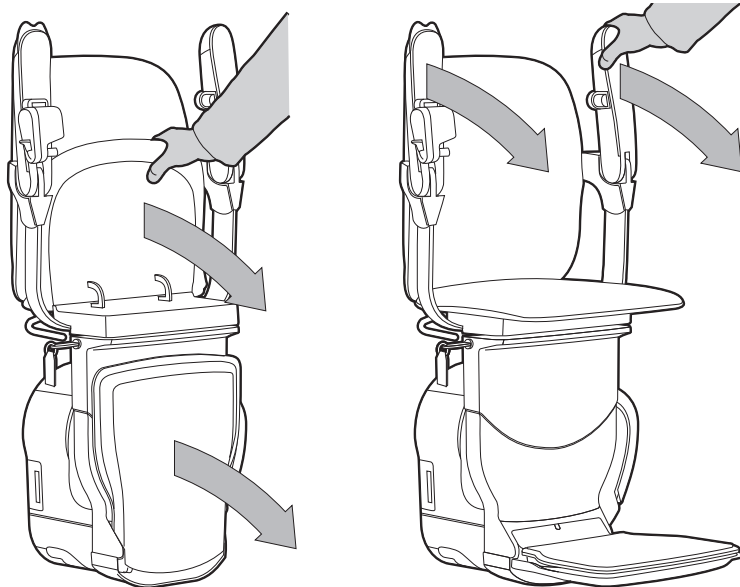

### SAFETY NOTICE

Stand with your feet clear of the footrest when you lower the seat.

# Using your stairlift

## Using the retractable rail (optional)

Your stairlift may be fitted with a retractable rail that will automatically raise so the rail does not obstruct any doorways or passageways.

### To use the retractable rail




The retractable rail will automatically move when you are traveling on the stairlift.

Please note that a light will flash and you will hear a series of beeps to warn you and others that the retractable rail is moving.

When you are at the bottom of the stairs, you will need to move the stairlift back up the stairs so that the retractable rail does not obstruct any doorway or passageways. This will also make sure that the stairlift is on charge.

Press and hold the 'up' arrow on the remote calling control until the retractable rail is fully raised.

Please note that if you leave the stairlift at the bottom of the stairs, it will start to beep. This is to warn you that your stairlift may cause an obstruction and that it is not on charge.

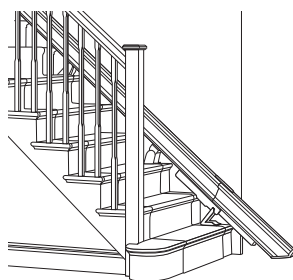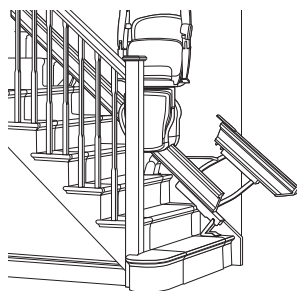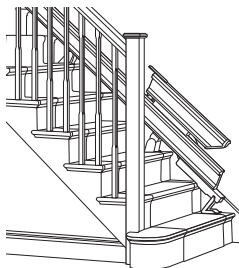

### SAFETY NOTICE

Make sure the area around the retractable rail is clear before moving the stairlift.


# Using your stairlift

## 3. Settle into the seat and start the stairlift

Make sure the on/off switch is in the off position and the power light is pulsing. Then, settle yourself into the seat, and put your seat belt on. When you are ready to travel up or down the stairs, switch the on/off switch to the on position, and use the arm control to start the stairlift.


### To use the seatbelt

Pull the seatbelt across your lap and click into the holder.

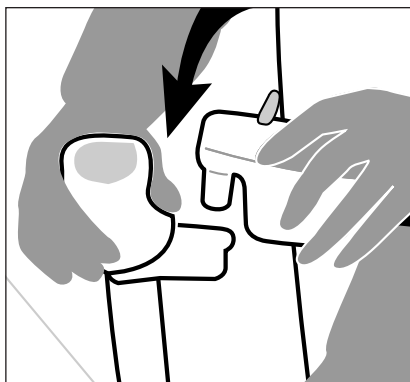

#### Immobiliser seatbelt (optional)

*If you have an immobiliser seatbelt, your stairlift will not start until you have put your seatbelt on.*

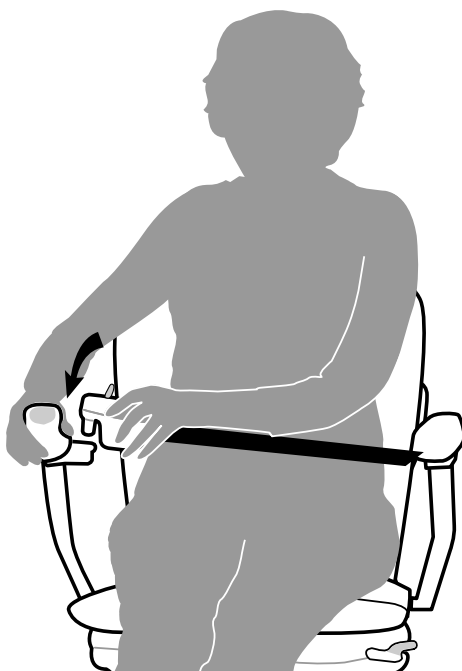

## SAFETY NOTICE


Always use the seat belt. Only undo it when you have reached the top or bottom of the stairs and are ready to get off. Keep your elbows tucked in close to your body when using the stairlift.

# Using your stairlift

## To use the armrest control

Move and hold the armrest control in the direction you want to travel – after a few seconds the stairlift will start. The delayed start is a safety feature.

When the stairlift has reached the top or bottom of the stairs, it will stop. If you release the control at any point, the stairlift will stop.

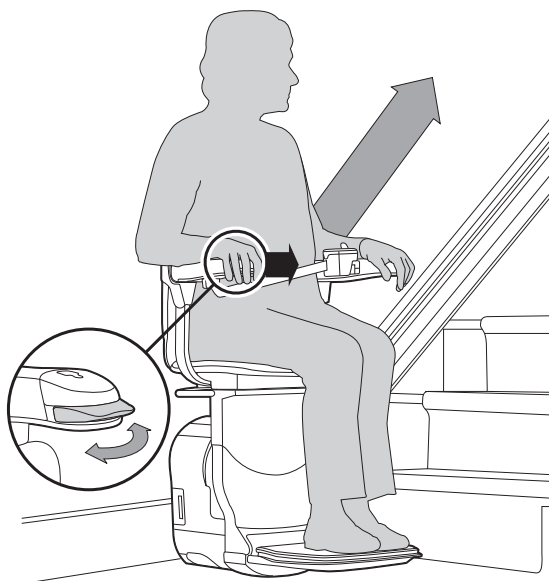

## Powered swivel seat (Optional)

If you have an powered swivel seat, you can continue to hold the arm control when you have come to the end of your journey and the seat will automatically swivel to the side. See page 13.

Please note that after the stairlift has stopped, there will be a few seconds' delay until the seat swivels.



## SAFETY NOTICE

Only one person should ever use the stairlift at one time.

# Using your stairlift

## 4. Swivel the seat


At the top of the stairs you can swivel the seat to help you get off the stairlift.

You should also swivel the seat to make it easier to get on the stairlift before you go back down.

### To swivel the chair using the manual swivel seat

Lift one of the swivel levers on either side of the seat and swivel the seat to the side.

You can only do this in the direction of the top of the stairs – this is a safety feature.

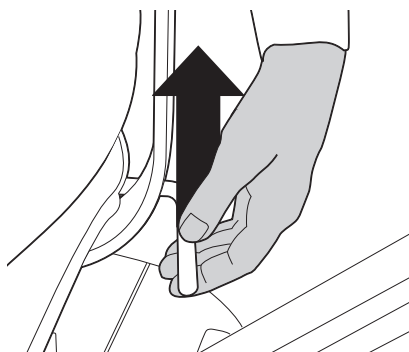


### To swivel the chair using the powered swivel seat (optional)

Push and hold the armrest control in the direction you want to swivel the seat.

### To swivel the chair using the two-way powered swivel seat (optional)

If you have a two-way powered swivel, you can swivel the seat at the top and bottom of the stairs.

*Please note that if you have selected a two-way swivel seat, the stairlift will only swivel in the downwards direction when it is at the bottom of the stairs.*

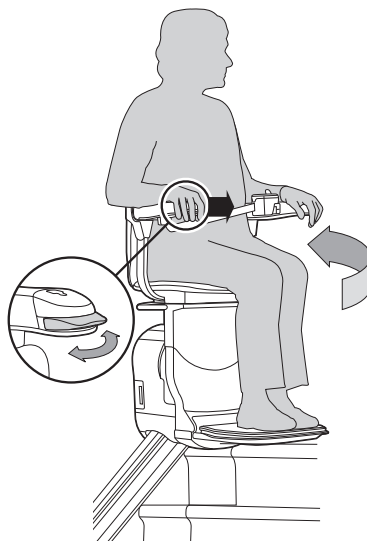

# Using your stairlift

## 5. Get off the stairlift


Make sure the on/off switch is in the off position and the power light is pulsing. Carefully, get off the stairlift and swivel the seat back to its traveling position. You can fold the chair away if you wish.

### To get off


**Unfasten the seatbelt.**

Slide the button towards the armrest and lift the clip out of the holder.

Carefully, get off the stairlift.

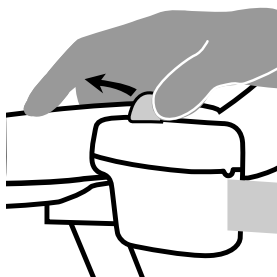

Swivel the seat back to its traveling position using either the manual swivel levers or, if you have a powered swivel, the remote calling controls. You can fold the chair away if you wish.

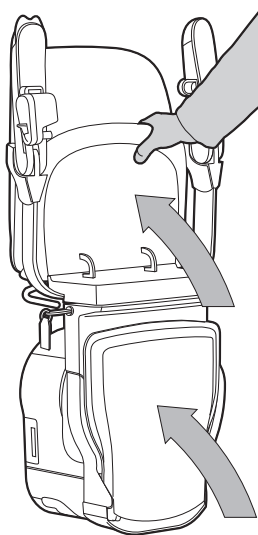

**For the 'HELP' section see page 18**

# Changing the battery on the remote control

The remote control batteries will last about one year with average use.

When you press one of the arrow buttons, the light around the buttons will tell you how much power is in the battery.

<b>Lights rotating</b>	<i>Batteries OK</i>
<b>Lights on constantly</b>	<i>Batteries low on power – change batteries as soon as possible</i>
<b>No light</b>	<i>Batteries have no power</i>




The lights are on constantly when the battery is low.

## To change the battery

Remove screw if fitted, slide down and open the battery cover, then insert the new batteries.

Each calling control takes three AA batteries.

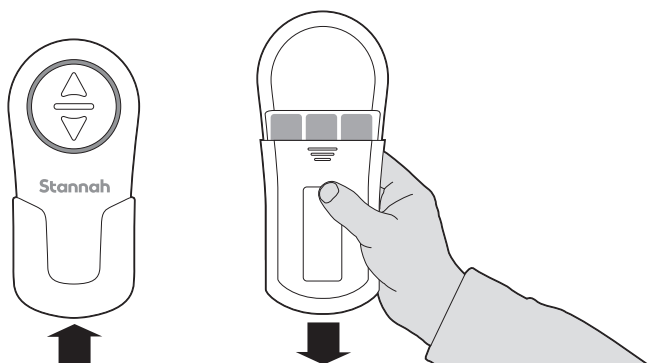

# Cleaning your stairlift

If there are any aspects of cleaning your stairlift that you are unsure about, please call your Stannah supplier.

## To clean the rail

Use a damp cloth to remove any dust or dirt.

## To clean the upholstery

Use a damp cloth to remove any dust or dirt.

**Do not use any solvent cleaners, bleaches, abrasives, synthetic detergents, wax silicone polishes or aerosol sprays without checking with your stairlift supplier first.**

## Recycling and disposal of your stairlift

Do not dispose of any part of this product as household waste. For more information about recycling and disposal of your stairlift, please contact your Stannah supplier, details on the back page.



## Safety recommendations

Follow these points to make sure you use your stairlift safely.

- Make sure no one else is using the stairlift before you call it using the remote calling controls.
- Only one person should ever use the stairlift at any time. **Do not exceed the weight limit, which is 350lbs**
- You can remove the power key to stop others from using your stairlift.
- Make sure the area around the retractable rail is clear before moving the stairlift.
- Stand with your feet clear of the footrest when you lower the seat.
- Never allow children to use the stairlift unsupervised.
- Always use the seat belt. Only undo it when you have reached the top or bottom of the stairs and are ready to get off.
- Never attempt to dismantle the stairlift.
- Keep your elbows tucked in close to your body when using the stairlift.
- If you are home alone when using your stairlift, we recommend carrying a cell phone, or some other communication device, so you can alert others if you need help getting off the stairlift.




# Help

If your stairlift does not move the problem is usually easy to fix. First try these steps:

## Check that the stairlift is turned on

Look at the red on/off switch on the armrest. It is lit by a blue LED light.

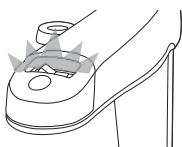

**LED off** - The master power switch is off - see page 9.

**LED pulsing** - Power key is not in correctly or on/off switch is in the 'off' position - see page 8

**LED on** - the stairlift is ready to move.

## Check the swivel seat

To travel, the swivel seat must be completely swiveled into the traveling position - see page 13.

## If the chair is all or part way up the stairs

The batteries may be getting low, but they should contain enough power to get the stairlift downstairs.

1. Remove and re-insert the power key - see page 8.
2. Move and hold the armrest control or one of the remotes in the downward direction. After a few seconds the stairlift should start.

*If the stairlift stops during the above, repeat steps 1 and 2 until you reach the bottom of the stairs.*

If steps 1 and 2 do not work, alert another person in the home and ask them to contact your Stannah supplier - see back cover for details.

If you are home alone when using the stairlift, we recommend carrying a cell phone or some other communication device, so you can alert others if you need help getting off the stairlift.

As a last resort, swivel the chair towards the top of the stairs, undo the seatbelt and exit up the stairs. Then call your Stannah supplier.



## Check the fault code

See page 19

# Help


## Optional retractable rail does not work

Please call your stairlift supplier. If the retractable rail is causing an obstruction, you can raise the rail manually using the underside of the rail.

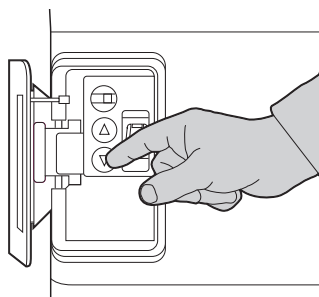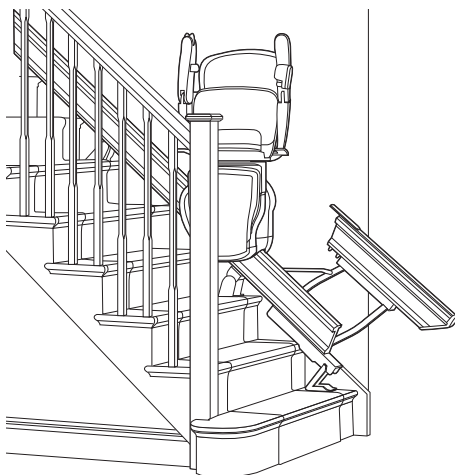

## Operating the emergency drive

Open access door on the carriage (the box below the seat) to reveal the stairlift controls. Press the top button to switch it to emergency mode. Press the control to drive the stairlift in the chosen direction. Once finished, press the top button again to switch it back to normal operation mode.

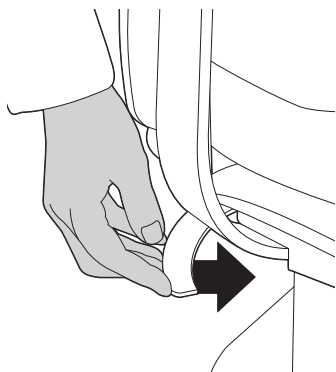

## To swivel the powered swivel seat

Switch off the master power switch-see page 6. Push both levers inwards. You can then swivel the chair. It needs to be in the traveling position before the stairlift will move up or down the stairs.

# Fault codes

A panel on the side of the stairlift shows a code to indicate the status of your stairlift. Before trying to diagnose a fault, try the steps on page 17. If you're still unable to solve the problem contact your stairlift supplier, whose details are on the back page of this guide.

Code	Why?	What should I do?
<b>-/1</b>	Charging / battery fault.	Check that the wall switch is on. Check that there has not been a power cut. If problem persists, call your Stannah supplier.
<b>2</b>	No faults found, normal operation mode.	Call your Stannah supplier if your stairlift does not work.
<b>3</b>	Obstruction detected around the footrest / carriage.	Check whether there is anything obstructing the safety edges. If there is, the stairlift is still on and can be moved in the opposite direction to allow you to remove the obstruction. You can then continue to go up or down the stairs.
<b>4</b>	Power key is not in correctly or on/off switch is in the 'off' position.	Check power key is in correctly and on/off switch is in the 'on' position - see page 8.
<b>5</b>	Downside arm not lowered.	Ensure downside arm is fully lowered. See page 9.
<b>6</b>	Immobilizer seatbelt is not on correctly (optional).	If you have an immobilizer seatbelt fitted then ensure that it is clicked into place - see page 11.
<b>7</b>	Seat is not in its travelling position.	Swivel seat back to its travelling position - see page 4.
<b>9/F</b>	Service Engineer required.	Call your Stannah supplier.

Contact Details

# Stannah

Stannah Stairlifts Inc, 20 Liberty Way, Unit A, Franklin, MA 02038, USA

**[www.StannahStairlifts.com](http://www.StannahStairlifts.com)**

Issue 2 September 2013



600463

English

Original Instructions

